



replacement parts industries, inc.

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028
E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

ADK122 LEVER FOOT CONTROL SERVICE KIT INSTALLATION INSTRUCTIONS

1. Turn off master switch. Remove screws from base plate. Remove foot control cover and retaining ring.
2. Completely disassemble foot control valve.
3. Inspect all parts and replace as needed. Be sure to clean and lubricate the Valve Stem (RPI Part #ADS106) and O-rings (RPI Part #RPO309 and RPO327) using Lubricant (RPI Part #RPL090) prior to reassembly. Refer to the diagram to the right to make sure all parts are properly located.
4. Install foot control cover and recheck the function of the foot control. (See "**Warning**" note to the right.)

