



replacement parts industries, inc.

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028
E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

"The Alternate Source™"


TUH043 & TUH044

WIRING HARNESS

WIRING DIAGRAM INSTRUCTIONS

1. Before installing the wire harness make sure plug is disconnected from the power source and that there is no pressure in the chamber.
2. If applicable, disconnect existing wire harness and connect new wire harness per diagram below. (Note: Connector modification may be necessary on some units.)
3. Check that all connections are correct and secure.
4. Apply power to the sterilizer.
5. Run a cycle to verify proper operation.

RPI Part #TUH043
Wiring Diagram

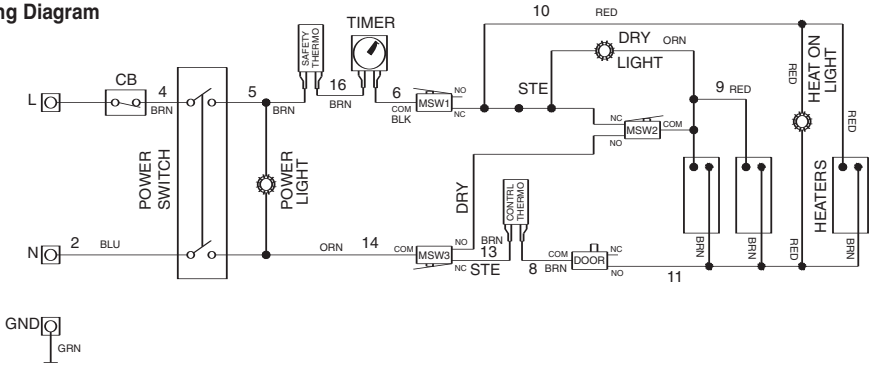

RPI Part #TUH044
Wiring Diagram

