



ADK245 FOOT CONTROL III SERVICE KIT INSTALLATION INSTRUCTIONS

1. Turn off the master switch and bleed the system of any trapped air.
2. Remove the screws from the base plate to remove the retaining ring and foot control cover.
3. Completely disassemble foot control valves, one at a time.
4. Replace all parts included in kit per diagram. (*Parts included in kit are shown in bold.*)
5. Be sure to clean and lubricate all pistons and o-rings, using High Temp Lubricant (RPI Part #RPL090) included in kit, prior to reassembly.
6. Refer to the diagram to make sure all parts are properly located.
7. **Warning:** Failure to replace the foot control cover before checking the function of the foot control may result in serious injury.
8. Install retaining ring, foot control cover and check the function of the foot control.

Warning: Failure to replace the foot control cover before checking the function of the foot control may result in serious injury.

