



replacement parts industries, inc.

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028
E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com


ADV244

MICRO VALVE INSTALLATION INSTRUCTIONS


To fit the A-dec Foot Control I, II, III & Lever Action Foot Control

1. Turn off the master switch and bleed the system of any trapped air.
2. Remove the screws from the base plate to remove the retaining ring and foot control cover.
3. Remove the screws that hold the 2-hole housing cover to gain access to the Foot Control Accessory Button and/or the Wet/Dry Toggle Valve.
4. Loosen the socket head set screw on the holder valve housing to release the Micro Valve.
5. Note the tubing position and remove the tubing from the old Micro Valve.
6. Reinstall the tubing onto the new Micro Valve (RPI Part #ADV244) at the same position as removed in Step #5.
7. Install the new Micro Valve into the holder valve housing. Adjust the height of the Micro Valve to insure proper function when toggle lever or actuator button is activated. Tighten the set screw.
8. Reinstall the 2-hole housing cover, trapping the Foot Control Accessory Button and the Wet/Dry Toggle Valve against the base plate.
9. See "Warning" note. Install retaining ring and foot control cover and check the function of the foot control.

FOOT CONTROL ACCESSORY BUTTON

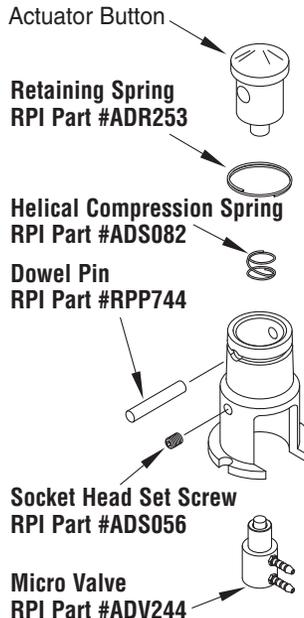

WET/DRY TOGGLE VALVE

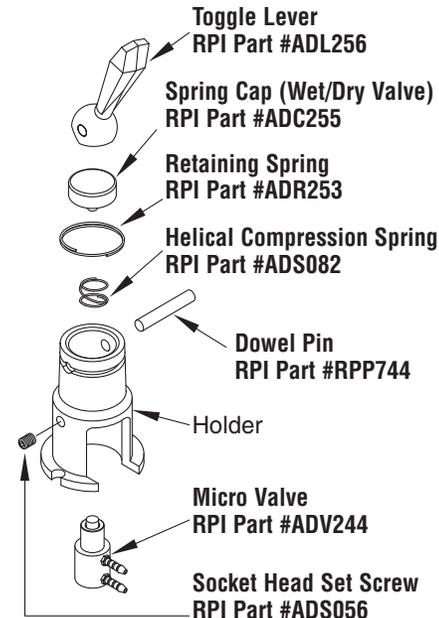

Warning: Failure to replace the foot control cover before checking the function of the foot control may result in serious injury.

ADV244 INS REV- (07/11)