


**EXTENDED LIFE GREEN SEALING GASKET
ONLY SUITABLE FOR USE WITH
SERIES 2100 Autoclaves**

**FROM SERIAL NO. AE01001 ONWARDS,
MANUFACTURED FROM 1ST APRIL 1992**

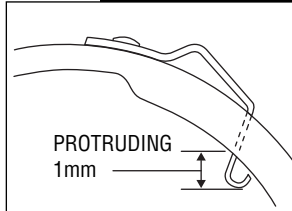


IMPORTANT: Before fitting new gasket, it is essential that the Gasket Offset Device spring setting is correct - as in diagram. This is an important safety device which ensures pressure cannot build up if the lid is incorrectly fitted. If in doubt about the setting, contact your supplier or service engineer.

Care and maintenance of your sealing gasket

It is essential that you carry out the following simple procedures on a weekly basis in order to get the best use out of your new gasket.

1. Remove the gasket from inside the lid and wash with warm, soapy water.
2. Rinse thoroughly, do not dry, use wet.
3. Replace in lid as follows: Insert gasket into Gasket offset Device area first. Tuck evenly under all lugs until gasket is seated as shown. It may appear slightly 'wrinkled' until used.
4. Never stretch, distort or fold gasket.
5. Never use any abrasive cleaning materials or lubricants.
6. Replace the gasket when it begins to show signs of leakage.
7. Should a new gasket leak, or a persistent leak develop, clean both the lid and body sealing surfaces with a 'Scotchbrite' scrubbing pad. (Work gently around surfaces - do not remove any metal - clean only). Rinse thoroughly but do not dry sealing surfaces. Do not use "Scotchbrite" inside the autoclave body.

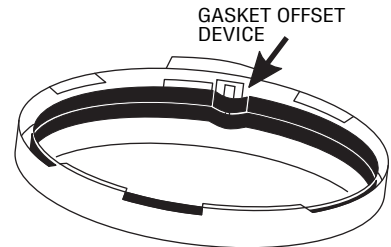

replacement parts industries, inc.

Call (800) 221-9723 • Fax (818) 882-7028 • order@rpi.com • www.rpiparts.com

To reorder, ask for **Door Gasket (RPI Part #KVG001)**

KVG001 LBL Rev-

**USE ONLY PRESTIGE MEDICAL APPROVED
SEALING GASKETS**

The manufacturer and supplier disclaim all liability for product failure due to use of incorrect gasket or incorrect setting of the Gasket Offset Device by the user.