



replacement parts industries, inc.

"The Alternate Source"®

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028
E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

MTA002 DISINFECTANT FILTER ASSEMBLY INSTALLATION INSTRUCTIONS

IMPORTANT

The OEM recommends that the Disinfectant Filter Assembly be changed every time the discard/load procedure is performed.

1. The used disinfectant filter assembly may contain residual liquid. Place an absorbant cloth or container under the disinfectant filter assembly before disconnecting it from the system.
2. Remove the used disinfectant filter assembly.
3. Inspect the female quick-connect fittings for dirt and debris and clean as necessary.
4. Note that the male quick-connect ends on the disinfectant filter assembly are color coded to ensure proper direction of liquid flow when installing the new disinfectant filter assembly.
 - Match the red (inlet side) male quick-connect fitting to the red female quick-connect fitting (from the reservoir). Install new disinfectant filter assembly. (The reservoir end female quick connect fitting may be white or beige on early DSD-201 & DSD-91E models.)
 - Now do the same for the grey quick-connect set (to the manifold assembly). Flow is from the reservoir to the manifold assembly.
5. Record the date of the filter replacement in the appropriate log.