



"The Alternate Source"™

# replacement parts industries, inc.

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028

E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

## PCK025 CONVERSION KIT INSTALLATION INSTRUCTIONS

**Use this conversion kit only with RPI's Heater (RPI Part #PCH022).**

- 1) Remove old non-grounded power cord and strain relief bushing.
- 2) Install new grounded power cord and strain relief bushing.
- 3) Check wiring and replace if needed.
- 4) Locate an area on the right rear side of auto-clave with ample clearance. Cut 1/2" hole and mount switch. Be sure there is clearance between switch terminals and bottom plate.
- 5) Wire per diagram to the right.

