



replacement parts industries, inc.

CALL (800) 221-9723 or (818) 882-8611 • FAX (818) 882-7028

EMAIL techsupport@rpiparts.com • WEBSITE www.rpiparts.com

ADR185, MAR001, PCR709, PCR710 & RCR608 REFLECTOR PROPER CARE INSTRUCTIONS

READ THIS WARNING BEFORE CLEANING REFLECTOR

- Do not clean reflector if it is warm or hot.
- Allow reflector to cool before beginning the cleaning process.
- Notify all persons who come in contact with reflector, that reflector will break if it gets wet while it is warm or hot.
- Do not use abrasives, chlorine or ammonia to clean the reflector.
- Do not soak the reflector in any kind of cleaning solution.
- Do not scrub the reflector.

Recommended cleaning cycle under normal usage: Approximately once per year

1. Use a clean, soft cotton cloth that has been soaked in isopropyl alcohol to wipe the reflector surface in one direction with vertical strokes. (Important: Do not clean reflector if it is warm or hot.)
2. Use a clean, soft cotton cloth to remove the isopropyl alcohol residue from the reflector. Wipe the surface in the same manner as used in Step #1.
3. If cleaning the back surface of the reflector, use a clean, soft cloth with a mild detergent. Wipe the surface in one direction with vertical strokes. Do not allow the detergent to come in contact with the front surface. (Important: Do not clean reflector if it is warm or hot.)