



**replacement parts industries, inc.**

*"The Alternate Source"™*

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028

E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

## **RCK123 AIR VENT BELLOWS KIT INSTALLATION INSTRUCTIONS**

- 1) Remove the defective bellows assembly from the water reservoir.
- 2) Attach one end of the copper tube with compression fittings to the port from which you just removed the defective bellows and tighten fitting.
- 3) Attach the new Air Vent Bellows (RPI Part #RCB089) to the open compression fitting on the copper tube and tighten fitting. Make note of the arrow on the new Air Vent Bellows showing the direction of the air flow.
- 4) Run a complete sterilization cycle to ensure that there are no leaks and the machine runs properly.

