



replacement parts industries, inc.

"The Alternate Source"™

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028

E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

RPB365

RIGHT ANGLE LIQUID TIGHT STRAIN RELIEF BUSHING WITH LOCKING NUT

INSTALLATION INSTRUCTIONS

When installing this part in the sterilizer, follow these instructions:

1. Insert power cord into Strain Relief Bushing (SRB). Pull cord through SRB to desired length, allowing enough excess at the end of the cable for convenient hook-up.
2. When satisfied with placement of the power cord inside the SRB, snap the two halves of the SRB together then tighten the Flex Nut section.
3. Insert the power cord and threaded section of the SRB through the mounting hole in the machine and secure in place with the Locking Hex Nut.

NOTE: Once the bushing is snapped into place, it is not recommended to un-snap it and re-use it.

If using this part for other applications:

The hole size requirements are as follows:

- Mounting hole diameter – 3/4" (.73")
- Threads fit 1/4" NPT or 1/2" UNF