



replacement parts industries, inc.

"The Alternate Source"®

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028
E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

RPC288

POWER CORD WITH CONNECTORS

USAGE INSTRUCTIONS

MIDMARK STERILIZERS

If using this Power Cord to fit Midmark® • Ritter® Ultraclave® M9 & M11 Sterilizers:

- Use the enclosed 1/4" Non-Insulated High Temperature Quick Connect Terminal (RPI Part #RPT029) on the Overtemp Thermostat.
- Use the enclosed 1/4" Insulated Quick Connect Terminal (RPI Part #RPT015) on the PC Board.
- Use the enclosed #8 Ring Terminal (RPI Part #RPT006) for grounding.

PELTON & CRANE STERILIZERS

If using this Power Cord to fit Pelton & Crane Delta Series Sterilizers:

- No Terminals are used on Pelton & Crane sterilizers.
- Cut and strip wires as required.

SCICAN AUTOCLAVES

If using this Power Cord to fit SciCan® STATIM® 2000 & 5000 Cassette Autoclaves:

- Use the enclosed (2) 3/16" Fully Insulated Quick Connect Terminals on the Power Switch.
- Use the enclosed #10 Ring Terminal (RPI Part #RPT007) for grounding.