



replacement parts industries, inc.

"The Alternate Source"®

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028

E-MAIL: techsupport@riparts.com • WEBSITE: www.riparts.com

SDP064

VAPORIZER PLATE

INSTALLATION INSTRUCTIONS

CAUTION: *Always wear eye protection and gloves when removing or replacing the Vaporizer Plate.*

WARNING



The Vaporizer Plate must be installed correctly to ensure it performs as it was intended. Improper installation or "Not" using the Vaporizer Plate may result in hydrogen peroxide condensation remaining on the load after a complete cycle. Coming in contact with residual hydrogen peroxide can result in injury.

Removing the old Vaporizer Plate:

Locate the Vaporizer Plate in the top of the electrode assembly. While wearing gloves, use your fingers to squeeze the plastic plate just enough to free the sides from the electrode opening, then pull down and out. Discard the used Vaporizer Plate as required by law, and according to hospital, local, state and federal regulations.

Installing a new Vaporizer Plate:

Angle the Vaporizer Plate just enough to catch the hole in the electrode assembly. Squeeze the sides of the plate slightly and press it up and back until you feel it snap into place. Double check to ensure the Vaporizer Plate is installed correctly.

NOTE: It is recommended the Vaporizer Plate be replaced every 30 days or 145 cycles, whichever comes first.