



replacement parts industries, inc.

"The Alternate Source"®

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028
E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

SSF012 STERILE FILTER (A1525) IMPORTANT INSTALLATION NOTICE

*The following instructions must be followed
for proper installation of the RPI sterile filter.*

1. Set the DIP switch to the following settings:
 - No. 1 = ON
 - No. 2 = OFF
 - No. 3 = OFF
 - No. 4 = OFF
2. Make sure the filter is pre-wetted. Either soak it in sterile water or run a second diagnostic cycle if the first cycle fails.