



**replacement parts industries, inc.**

*A Healthcare Components Group Company*

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028

E-MAIL: [techsupport@rpiparts.com](mailto:techsupport@rpiparts.com) • WEBSITE: [www.rpiparts.com](http://www.rpiparts.com)

## **VPA203 & VPA204**

### **MOISTURE ALARM ASSEMBLY INSTALLATION INSTRUCTIONS**

**WARNING:** Remove power before working on these units.

#### **INSTALLATION INSTRUCTIONS**

1. Unplug the Moisture Alarm Assembly cable from the control PCB Assembly.
2. Pull the Moisture Alarm Assembly out of the grommet in a downward and side to side motion and then discard.
3. Remove the old grommet and discard.
4. Clean the opening where the old grommet was located. Be careful of any sharp edges.
5. Install the new Grommet (RPI Part #VPG205) where the old grommet was removed.
6. Push the replacement Moisture Alarm Assembly up into the Grommet installed in Step 5, until a tight seal is created.
7. Plug the Moisture Alarm Assembly cable into the control PCB Assembly.
8. Restore power to the unit and turn the unit on. Check the tightness of the seal of the Moisture Alarm Assembly into the Grommet. If any air is being drawn in past the Moisture Alarm Assembly, push the Moisture Alarm Assembly up tighter into the Grommet.