



# PM CHECK LIST

## ***Sterilizer PM Kit (RPI Part #AMK234) for parts to fit Eagle® 10 & 10 Plus sterilizers***

PART	RPI PART #	STERILIZER PM TIP
<input type="checkbox"/> Door Gasket	AMG161	Remove the old door gasket. Clean the gasket groove and chamber face ring surface. For easy install of the new gasket, use the "star" or "North-South-East-West" method (ie. begin installing the gasket at the top of the door, then the bottom, then the right side, then the left side, and then everything in between making sure that the gasket fits flush without bumps or gaps).
<input type="checkbox"/> Water Guard	AMG172	Use CAUTION when removing or installing the water guard, the tab for the bottom shelf can be sharp. Remove the old water guard and clean off the remaining adhesive. To install the new water guard peel the backing off, align, and place it onto the chamber wall but not over the tab. The water guard must lie flat against the chamber wall.
<input type="checkbox"/> Filter Fill Line	PCF009	Remove all trays from the chamber. Remove the chamber filter. Clean chamber. Install new filter making sure that it is placed at the bottom and center of the chamber.
<input type="checkbox"/> Sterilizer	-	After replacing the PM parts: <ul style="list-style-type: none"><li>• Check the reservoir and clean if dirt or debris are present (Note: Debris is the number one reason for Steam Trap/Bellows failure).</li><li>• Refill reservoir with distilled water only.</li><li>• Conduct a temperature check using a Max Register Thermometer (RPI Part #RPT113).</li><li>• Conduct a pressure check to identify any leaks that may need to be addressed.</li></ul>



**replacement parts industries, inc.**

CALL (800) 221-9723 or (818) 882-8611 • FAX (818) 882-7028

E-MAIL [order@rpi.com](mailto:order@rpi.com) • WEBSITE [www.rpiparts.com](http://www.rpiparts.com)

The above parts are manufactured by Replacement Parts Industries, Inc. to fit Amsco equipment.

All product names used in this document are trademarks or registered trademarks of their respective holders. AMK234INS REV A (12/10)