



PM CHECK LIST

Sterilizer PM Kit (RPI Part #MIK134)

for parts to fit M7 & 7 sterilizers

PART	RPI PART #	STERILIZER PM TIP
<input type="checkbox"/> Door Gasket	RCG019	Remove the old gasket. Using a tooth brush clean the gasket channel in the chamber and clean door surface. Gasket Installation Tip #1 - Lubricate gasket with soapy water before inserting into gasket channel. Gasket Installation tip #2 - For easy installation of the new gasket, use the "South-East-West-North" method ((i.e. begin installing the gasket at the top of the door, then install the bottom, then the right side, then the left side, and then everything in between making sure that the gasket fits flush without bumps or gaps).
<input type="checkbox"/> Chamber Filters (Mesh Style)	MIF062	Most of the older sterilizers may not have chamber filters at all. Remove all trays and the rack from chamber. Clean the chamber, then remove the old chamber filters if present. Install the new filters. Reinstall rack and trays.
<input type="checkbox"/> Sterilizer	-	After replacing the PM parts: <ul style="list-style-type: none">• Check the reservoir and clean if dirt or debris are present (Note: Debris is the number one reason for Steam Trap/Bellows failure).• Refill reservoir with distilled water only.• Conduct a temperature check using a Max Register Thermometer (RPI Part #RPT113).• Conduct a pressure check to identify any leaks that may need to be addressed.



replacement parts industries, inc.

CALL (800) 221-9723 or (818) 882-8611 • FAX (818) 882-7028

E-MAIL order@rpiparts.com • WEBSITE www.rpiparts.com

The above parts are manufactured by Replacement Parts Industries, Inc. to fit Midmark®/Ritter® equipment.
All product names used in this document are trademarks or registered trademarks of their respective holders. MIK134INS REV A (12/10)